MercuryMedical **EZflow MAX[®]**
Continuous Nebulizer (en)

11300 - 49th Street North
Clearwater, Florida 33762-4807 USA
www.mercury-med.com

INTENDED USE:
To be used to convert liquid medication into aerosol for inhalation by patient.

CAUTION:

- Federal law (USA) restricts this device to sale by or on the order of a physician.
- Reuse of this device may pose a risk of cross-contamination and the device may not perform as intended.
- The medicinal product which the device is intended to administer must be approved and used under the specific direction of a physician.
- While using supplemental oxygen, do not allow smoking or use unit near sparking equipment, open flame, oil or other flammable chemicals.
- Performance information may not apply to drugs supplied in suspension or high viscosity form.
- Disposal of this device should be handled in accordance with hospital protocol.
- This device is assembled with PVC components which include low levels of a phthalate as indicated. Consideration should be used when treating children or pregnant or nursing women.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

INSTRUCTIONS FOR CONTINUOUS NEBULIZATION:

1. Remove EZflow MAX[®] from packaging.
2. Connect EZflow MAX[®] to a flowmeter using oxygen supply tubing.


CAUTION:

- Be sure all connections are secure.
- Pour prescribed medication into nebulizer reservoir.
- Adjust flowmeter to desired flowrate of 6 or 10 L/min. Nominal output is 12 mL/hour at 6 L/min and 16 mL/hour at 10 L/min. Check the orifice of the nebulizer and clear any obstructions.

CAUTION:
- The listed output is nominal value only, actual output may vary depending on device, accuracy of flowmeter, and the existence of tubing connection leaks. Follow up output checks are recommended and flowmeter adjustments may be necessary.

5. Connect aerosol mask directly to the outlet of the EZflow MAX[®]. Place aerosol mask on patient and ensure a good mask seal.
6. Check liquid consumption rate (output) every hour.
7. Increase or decrease the nebulizer flow rate by small increments to obtain desired actual output.

EZFLOW MAX[®] CONTINUOUS NEBULIZER SETUP:

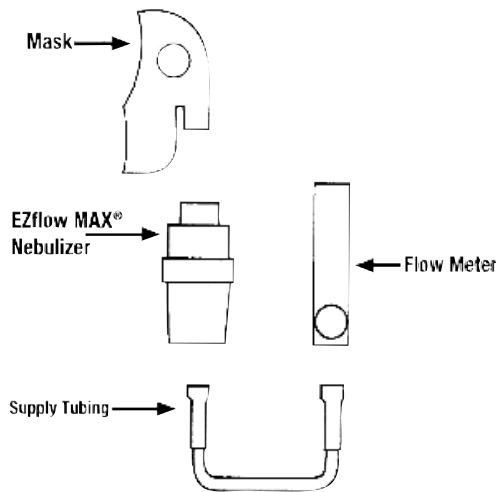SPECIFICATIONS:
Sample Guidelines for Preparing Rx Dosing:

5mg (1cc/Hr.)				10mg (2cc/Hr.)			
Time	Output @ 6Lpm	Albuterol in cc's	Saline Mixture	Time	Output @ 6Lpm	Albuterol in cc's	Saline Mixture
1 Hr	12cc	1cc	11cc	1 Hr	12cc	2cc	10cc
2 Hr	24cc	2cc	22cc	2 Hr	24cc	4cc	20cc
@ 10Lpm Flow				@ 10Lpm Flow			
1 Hr	16cc	1cc	15cc	1 Hr	16cc	2cc	14cc
1.5 Hr	24cc	1.5cc	22.5cc	1.5 Hr	24cc	3cc	21cc

15mg (3cc/Hr.)				20mg (4cc/Hr.)			
Time	Output @ 6Lpm	Albuterol in cc's	Saline Mixture	Time	Output @ 6Lpm	Albuterol in cc's	Saline Mixture
1 Hr	12cc	3cc	9cc	1 Hr	12cc	4cc	8cc
2 Hr	24cc	6cc	18cc	2 Hr	24cc	8cc	16cc
@ 10Lpm Flow				@ 10Lpm Flow			
1 Hr	16cc	3cc	13cc	1 Hr	16cc	4cc	12cc
1.5 Hr	24cc	4.5cc	19.5cc	1.5 Hr	24cc	6cc	18cc

EXAMPLE:
To deliver 10 mg/hour of 5 mg/mL medication at 10L/min; 1.5 Hour Duration: Place 3 mL of medication + 21 mL of Saline into nebulizer reservoir and run for 1.5 hours.

Rx Do Not Reuse
LATEX By Prescription Only
NON STERILE Not Made with Natural Rubber Latex
PHT DINP
MR MR Safe

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