


# Bili-Bonnet

**Latex Free-No adhesives**

How to put the patented Bili-Bonnet phototherapy mask on correctly

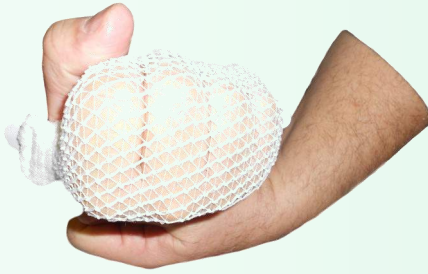

Remove from package, then stretch gauze and pull over fist to create a bonnet.



Stretch bonnet over baby's head to go from the nape of the neck to the forehead.



Attach velcro tab to one side. Note, the mask has a 1 inch area for better adjustment & velcro tabs are rounded .



Repeat with tab on other side. The mask should be snug & secure but not tight.




Ensure the netting is opened up all the way over the head to allow light to penetrate.



Now once again stretch netting down to the nape of the neck and up to the forehead.



**Most Important Step**  
Once all is in place and secure, unhook and reattach the velcro tabs to pull up/down/midline as head shape dictates.

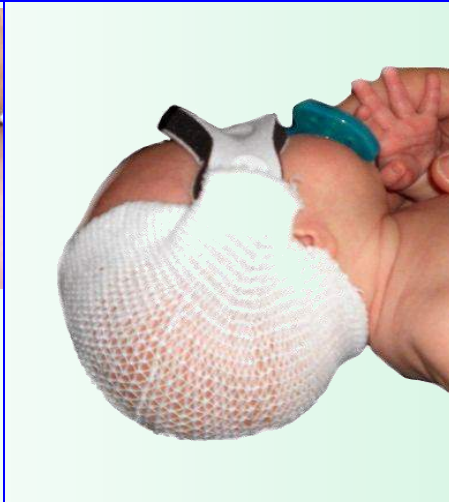

The end result is a mask that stays on better giving the nurse more time for other things. Protects the eyes better & allows more light to the head.

**Take time to save time !**