


Procedural Oxygen Mask™


# MASK ASSEMBLY INSTRUCTIONS

## ALL POM™ SERIES INCLUDE UNIVERSAL O<sub>2</sub> ADAPTER

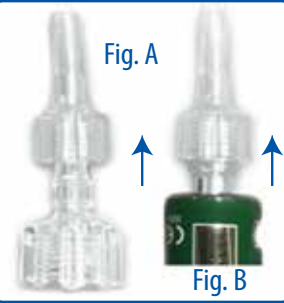


MEDIUM CONCENTRATION

### POM 1003™ EZ SERIES

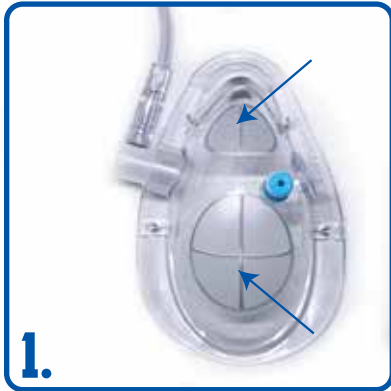

1.

Choose oral or nasal membrane port, Then push finger from inside mask through the EZ tear membrane. Fully separate connecting edges within membrane.

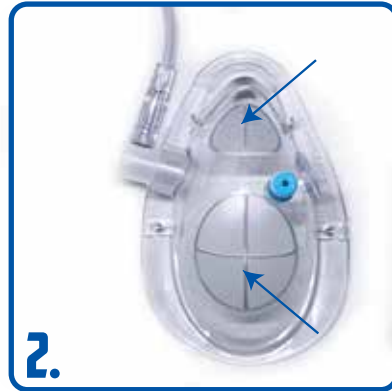

2.

#### PRE-ASSEMBLED

Universal O<sub>2</sub> line attached to medium concentration adapter



3.

First, unscrew blue cap. Then, securely attach [Leur-Lok] EtCO<sub>2</sub> sample line to threaded EtCO<sub>2</sub> adapter port on mask.



4.

Make sure Universal O<sub>2</sub> and EtCO<sub>2</sub> sample lines are functioning properly.

HIGH CONCENTRATION

### POM 1001 LT™ SERIES



1.

Choose oral or nasal membrane port, Then push finger from inside mask through the EZ tear membrane. Fully separate connecting edges within membrane.



2.

Choose High or Medium Concentration

Insert Medium or High Concentration non-rebreather adapter into Universal O<sub>2</sub> port on mask. Make sure non-rebreather adapter is fully seated and flush with mask. Make sure one-way valve or non-rebreather adapter is working properly.



(If High Concentration, push in until snaps in.)



Securely Attach Universal O<sub>2</sub> line to either **medium** or **high** concentration adapter.



3.

First, unscrew blue cap. Then, securely attach [Leur-Lok] EtCO<sub>2</sub> sample line to threaded EtCO<sub>2</sub> adapter port on mask.



4.

Make sure Universal O<sub>2</sub> and EtCO<sub>2</sub> sample lines are functioning properly.