

TnT Tourniquet



TnT-Tender and Tiny Tourniquet
Dynamite product!

Eliminate dirty rubber bands with
Latex free & Anti-microbial Bands
Packaged individually for single patient use
Help to prevent nosocomial infections
Best practice always "Do No Harm"



Order #	Product Description	Pkg Qty
TnT Tourniquet		
IX-11TA	Preemie Tourniquet	6
IX-11TH	Head Tourniquet	6

MercuryMedical[®]

11300 - 49th Street North • Clearwater, Florida 33762-4807 USA
ORDERING INFORMATION: 800.835.6633 or 727.573.0088
Fax: 727.573.6040

www.mercurymed.com

TnT Tourniquet



Tender N Tiny

- Perfect for the tiny preemie
- Latex Free
- **Antimicrobial**
- Individually packaged
- Comes 6 individual in a bag of 6
- Round tourniquet for head also



Small Beginnings
The Tiny Baby Company

17229 Lemon Street, #92
Hesperia, CA 92345 USA
www.small-beginnings.com
800-676-0462

REF Catalog # 11TA

TnT Tourniquets-Preemie size

Instructions for Use- Tender and Tiny (TnT) Tourniquets
1. Remove from package and inspect product to ensure there are no defects.
2. Stretch tourniquet a few times to check flexibility.
3. Place tourniquet under desired extremity.
4. Lift both sides to create pressure (not too tight).
5. Gently tie like a bow leaving one loop under the tourniquet to secure.
6. To remove pull one end and the tourniquet will release.

Wipe clean with alcohol, can be re-used on the same patient.
Latex-Free and Antimicrobial

LOT 19211TA-01 QTY: 6

Manufacture Date:
2017-06-22

Shelf Life:
5 Years

