# FLCWSAFE II®

DISPOSABLE CPAP SYSTEM

(en)

Mercury Medical®  
11300 - 49th Street North  
Clearwater, Florida 33762-4807 USA  
www.mercurymed.com

**DESCRIPTION:**  
The Mercury Flow-Safe Continuous Positive Airway Pressure (CPAP) device is a respiratory aid intended for use with a facemask and gas supplying device to elevate pressure in the patient's lungs.

**CONNECTIONS:**

- Standard oxygen tubing nipple
- Patient connection ISO 5356-1 - 15mm taper female and 22mm taper male

**INDICATIONS FOR USE:**  
The Mercury Flow-Safe CPAP device is intended to provide CPAP to spontaneously breathing patients in the hospital and pre-hospital environment.

**WARNINGS:**

- Do not allow smoking or use unit near sparking equipment, open flame, oil or other flammable chemicals.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
- CAUTIONS:**

  - Federal (USA) law restricts this device to sale by or on the order of a physician.
  - Do not clean, soak, rinse or sterilize.
  - Reuse of this device may pose a risk of cross-contamination and the device may not perform as intended.
  - In the event of undesirable flow rate from oxygen source, simply remove the device and place on supplemental oxygen per protocol.
  - Use of the Flow-Safe with non-back pressure compensated flow devices may affect input gas liter flow. Always verify delivered CPAP pressure on manometer.

**CONTRAINdications:**

- Respiratory Arrest
- Unconscious
- Cardiogenic Shock
- Pneumothorax
- Facial Anomalies
- Facial Trauma

**DIRECTIONS FOR USE:**

1. Connect O<sub>2</sub> tubing nipple to gas source.
2. Secure the face mask snugly to patient's face using head harness.
3. Slowly increase gas flow to 6 or 8 lPM. Check mask fit to patient and device connections for leaks.
4. Adjust the flowmeter until desired pressure is obtained. Flow of 12 - 14 lPM is required to reach CPAP pressure of 8.5 - 10 cm H<sub>2</sub>O.
5. Do not exceed 30 lPM.
6. Patient SaO<sub>2</sub> should be monitored using a pulse oximeter.

**MEASURING PRESSURE:**

- Pressure relief limits maximum CPAP pressure to 25 cm H<sub>2</sub>O @ 25 lPM.
- Do not exceed pressure limit of manometer (25 cm H<sub>2</sub>O).
- Manometer accuracy ± 3 cm H<sub>2</sub>O up to 15 cm H<sub>2</sub>O and ± 5 cm H<sub>2</sub>O over 15 cm H<sub>2</sub>O.

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© Zarejestrowany w USA znak towarowy firmy Mercury Enterprises, Inc.