(en)

(da)

(nl)

(fr)

(de)

(el)

(it)

(no)

Mapleson Anesthesia Breathing Circuit

Single-Patient-Use

INDICATIONS FOR USE:

Mercury Medical Mapleson Anesthesia circuits divides non-rebreathing circuits into functionally similar groups, on the basis of the fresh gas flow required to prevent rebreathing and the ease with which intermittent positive pressure ventilation may be performed.

SUGGESTED USE:

- Attach the circuit following standard operating procedures.
- Prior to use on patient: ensure that all connections are secure. Fully extend tubing (if tubing is expandable) and add pressure to check for any leakage or circuit occlusion.

WARNING:

- Do not soak, rinse, wash, gas or steam sterilize!
- When using with a pressure manometer for airway pressure monitoring, do not connect to pressure sources exceeding the limits of the manometer.
- When using supplemental oxygen, do not allow smoking or use unit near sparking equipment, open flame, oil or other flammable chemicals.
- If any portion of the product becomes occluded, discontinue use and replace product immediately with new unit.

CAUTION:

- Federal Law (USA) restricts this device to sale by or on the order of a physician.
- Reuse of this device may pose a risk of cross-contamination and the device may not perform as intended.
- This device is assembled with PVC components which include low levels of a phthalate as indicated. Consideration should be used when treating children or pregnant or nursing women.

MR CONDITIONAL:

Static magnetic field of 3.0 Tesla or less
Spatial Gradient of 3000 gauss/cm or less

Patent # US 8,522,618 B1; US D831,202S where applicable

Assembled in USA with foreign components 7/2020 #83-900-0284 Rev. 8

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