INDICATIONS FOR USE:
The Neo-Tee T-piece Resuscitator is a gas powered emergency resuscitator intended to provide emergency respiratory support by means of a face mask or a tube inserted into a patient's airway. It is intended for use with neonates and infants weighing less than 10kg (22lb).

ACCESORIES:
Round Silicone Masks Sizes: 35mm, 42mm, 50mm, 60mm or 76mm (15mm port)
Anatomical Silicone Mask Sizes: 37mm or 43mm
Anatomical Cushion Mask Sizes: Neonatal and Infant

- WARNINGS:**
- Incorrect operation of this device can be hazardous. Circuit should not be used without the manufacturer's instructions.
 - Do not use in the presence of flammable materials and ensure that no sources of fire are present while the device is in use. Fire hazards are possible in oxygen enriched environments.
 - Users should uncover the PEEP knob hole as soon as the pressure is delivered. Failure to do so will result in an undesired leak, which may prevent adequate ventilation.
 - Make sure you have a good mask seal and proper size mask.
 - Do not remove patient in an extended breath hold.
 - The recommended gas flow range is from 5 - 15 LPM. DO NOT EXCEED 15 LPM.
 - Remove Red protective cap from the T-piece circuit prior to connecting the T-piece to the mask and before use. See Figures 1 & Figure 2.
 - Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user's practice is established.

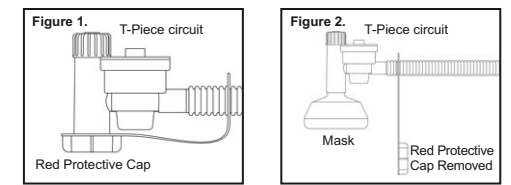
Figure 1. T-Piece circuit Figure 2. T-Piece circuit

CAUTIONS:

- Never start to begin resuscitation. If a Neo-Tee system is not immediately available, use mouth-to-mouth or the event of an undesirable flow rate from the gas source, see your department manual or AHA guidelines for other accepted resuscitation procedures.
- US Federal Law restricts the device to sale by or in the order of a physician.
- To be used only by persons who have been trained in infant/child resuscitation.
- Device to be used only in conjunction with a pre-use verification check to ensure that correct ventilation pressures will be delivered to the patient. Discard any device that fails a pre-use check.
- Oxygen concentration should be monitored at all times using an oxygen analyzer.
- Adjusting input flow will affect PIP and PEEP. Device can achieve high PEEP, always verify with the manometer.
- Do not attempt to disassemble any part of this circuit.
- Use of the device may result in cross-contamination and the device may not perform as intended.
- This device is assembled with PVC components which include low levels of a phthalate as plasticizer. Consideration should be given to treating children or pregnant or nursing women.

PRE-USE CHECK:

- Connect oxygen to flowmeter. If assembly does not include oxygen tubing, use suit & barb connector adaptor if required (See Figures 3 & 4).
- Adjust the flow on the flowmeter between 5 and 15 LPM.
- Connect an intratracheal test lung to the patient port of use Red protective cap for testing pressures if test lung is not available.
- Check the inspiratory pressure, by occluding the PEEP knob hole with the Peak Inspiratory Pressure (PIP) knob on the controller to test the Peak Inspiratory Pressure (PIP) as indicated by the manometer. Adjusting input flow rate will affect PIP.
- Release Shunt (or Index) finger from PEEP knob hole and adjust variable PEEP. Once to desired value as indicated on the manometer. Adjusting input flow rate will affect PEEP. Device can achieve high PEEP, always verify with the manometer.

PRE-USE CHECK (CIRCUIT ONLY):
Refer to the manufacturer of the resuscitator control units operating instruction for pre-use check.

DIRECTIONS FOR USE:

- Connect patient circuit to mask and place over patient's mouth and/or nose, or connect patient circuit to mask and place around patient's airway or endotracheal tube.
- Resuscitate by placing and removing thumb on the Index finger from the PEEP knob hole to allow inspiration and expiration at the desired breath rate, following hospital protocol.

DIRECTIONS FOR USE (CIRCUIT ONLY):
Refer to the manufacturer of the resuscitator control units operating instruction for pre-use check.

FOR PIP PRESSURES HIGHER THAN 40 CM H₂O USING RED OVERRIDE BUTTON (IF EQUIPPED):
If PIP higher than 40 cm H₂O required, turn adjustable pressure knob clockwise until pressure reading on the manometer. Adjusting input flow rate on blue pressure knob passes the Red button.

FOR PIP PRESSURES HIGHER THAN 40 CM H₂O USING RED OVERRIDE BUTTON (IF EQUIPPED):
If PIP higher than 40 cm H₂O required, turn adjustable pressure knob clockwise until pressure reading on the manometer. Adjusting input flow rate on blue pressure knob passes the Red button.


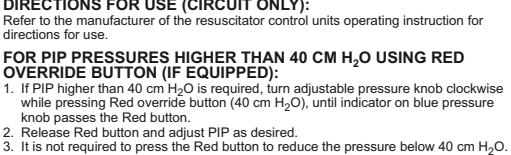
Figure 3. (Non In-line version) Figure 4. (In-line version)

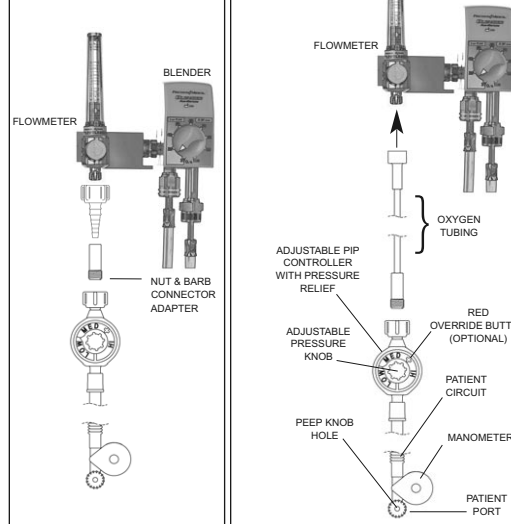PATIENT CIRCUIT FEATURES:

- Variable PEEP manometer
- Variable PIP controller (Does not apply to circuit only)
- Fixed Safety relief valve (40 or 60 cm H₂O) (Does not apply to circuit only)
- Integral manometer (40 or 60 cm H₂O) (Optional for all circuits)
- 30 cm (12 inches) corrugated tubing (183 cm (72 inches) for Non In-line version)
- 15mm female ISO patient connection
- CGA 120 Oxygen DISS standard connectivity (Does not apply to circuit only)
- (Optional) PIP override controller with red button (Does not apply to circuit only)

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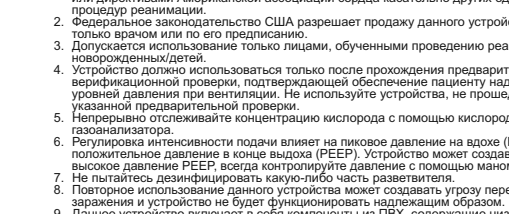
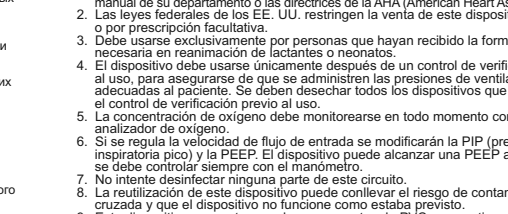
PRECAUTIONS:
1. Incorrect use or misuse of this device can be dangerous. Do not use the circuit without a pressure relief valve.
2. Do not use the device in the presence of flammable materials and ensure that there are no ignition sources present while the device is being used. There is a risk of fire occurring due to the incorrect use of the device.
3. Do not use the device near heat sources that could damage the device.
4. Adjust the gas flow to ensure that the mask is suitable for the patient's size.
5. Do not leave patients without supervision when using the device.
6. Do not leave patients without supervision when using the device.
7. Remove the red protective cap from the T-piece circuit before connecting the T-piece to the mask.
8. Incorrect or improper use of this device can be dangerous. The circuit should not be used without the safety valve.
9. Incorrect or improper use of this device can be dangerous. The circuit should not be used without the safety valve.
10. Incorrect or improper use of this device can be dangerous. The circuit should not be used without the safety valve.

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